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Product

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Batchnummer / Batch no. / Batchnummer /
Número de lot

Patient Information Leaflet Revamil

The product

Revamil hydrophilic wound gel
18 g honey for the treatment of wounds

Product description

Revamil is a wound gel that consists of pure honey, which has been produced under controlled conditions. The carefully controlled production process guarantees that Revamil does not contain any traces of pesticides and that the same quality can be reproduced consistently. The honey used to make Revamil contains a high proportion of honey enzymes such as glucose oxidase. Revamil should be used in combination with semi-occlusive wound dressings that guarantee a moist wound environment and that can be attached in the normal way. The highly concentrated sugars cause some of the wound exudate to mix with Revamil gel. The resulting layer helps to strengthen the antibacterial protection provided by the dressing.

Instructions for use

1. Before applying Revamil gel first clean the wound.
2. Spread a thin layer of Revamil on the secondary wound dressing.
3. Revamil can also be applied directly to the wound and then covered with a secondary dressing.
4. Attach the secondary dressing in the normal way.
5. Depending on the nature of the wound, replace the dressing every day or every two days.

Indications

Revamil wound gel can be used to treat chronic sores (bed sores and different kinds of ulcers), infected wounds, acute wounds, oncological wounds and minor (first- and second-degree) burns.

Contra indications

- Do not use Revamil if the patient is known to be allergic or hypersensitive to honey.

Side effects

- In rare cases Revamil wound gel can cause a momentary stinging sensation.
- If the patient experiences unexpected side effects after using Revamil, contact the distributor.

Safety precautions

- Do not use the gel if the tube is damaged.
- Always replace the top on the tube after use.
- Before applying Revamil, wet the wound with 0.9% NaCl.
- Do not allow the tube to come into direct contact with the surface of the wound.
- Use the contents within 3 months after opening tube.
- Do not treat different patients with the same tube.
- Do not treat a wound for more than 30 days consecutively.

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Bfactory B.V., Wageningen (NL)
Pharmadeal B.V.
Postbus 98, 9640 AB Veendam (NL)