Vetramil®

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Product

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Instructions for use

Product

Vetramil® wound ointment
10 g / 30 g wound ointment with honey and essential oils for wound treatment on animals.

Indication

Vetramil® stimulates healing of minor and large wounds, fights wound infections and can be applied on all domestic animals and pets. Vetramil® soothes and protects wounded and irritated skin.

Product Performance

Vetramil® is a honey-based ointment with essential oils. The ointment softens and protects the dry skin. Vetramil® contains honey with a very high enzyme content. These enzymes have an anti-bacterial effect and rapidly clean infected wounds. The low pH creates a natural and healing wound environment. Essential oils stimulate repair of the damaged skin and enhance the disinfecting activity. There is no waiting period after usage of Vetramil®.

Instructions for application

1. If necessary, clean the wound with water and clean tissue
2. Apply Vetramil® to the wound once a day
3. Massage well onto the irritated skin
4. Do not cover small wound
5. Cover large wounds with a secondary dressing and fixate the dressing with a bandage
6. The treatment can stop when the wound is closed

Ingredients

Ingredients: petrolatum, cetearyl alcohol, sodium lauryl sulfate, paraffin, isopropyl myristate, Thymus serpyllum, Lavandula angustifolia, Melaleuca quinquenervia, Pogostemon cablin

Conditions for storage

Storage at Room temperature

Precautions

- Do not use the ointment if the tube is damaged
- Close the tube after use
- Avoid direct contact between the tube and the wound
- Use the contents within 3 months after opening

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